

Did you know

you can temporarily change your SmartGuard™ feature target?

The SmartGuard™ technology targets are:

100mg/dL (default) 110mg/dL 120mg/dL

The target can be temporarily changed to 150 mg/dL for exercise or other times when less insulin is needed. Consult a healthcare professional before using a Temp Target.

When using SmartGuard™ technology, the Temp Target can be accessed from the SmartGuard™ menu. The duration can be set from 30 minutes up to 24 hours each time.

Best practice:

Set the Temp Target 1-2 hours before activity, keep it on during, and stop at the end.

Notes:

- Auto corrections are not delivered during a Temp Target.
- After the programmed time, the Temp Target will automatically end and the programmed SmartGuard™ target will be used. The Temp Target can also be canceled at any time.

To set a Temp Target:

1. From the Home screen, press **Select**, and then **SmartGuard**.
2. Select **Temp Target**.
3. Use the **Up** and **Down** arrows to set the Duration.
4. Select **Start**.



The screen shows **Temp Target Started**, and then returns to the Home screen, where a banner shows the remaining Temp Target time.

When the Temp Target time runs out, the banner disappears from the Home screen.

To cancel Temp Target:

1. From the Home screen, press **Select**, and then **SmartGuard**.
2. Select **Cancel Temp Target**.

The Temp Target will be canceled, and the Home screen will appear, with no Temp Target banner.

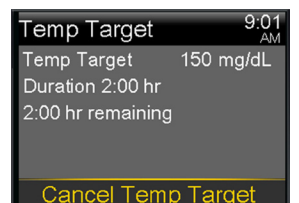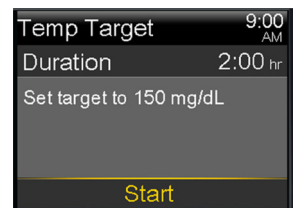

Did you know

you can temporarily change your SmartGuard™ feature target?

Use Suspend All Delivery if you need to stop all active basal and bolus insulin deliveries. You should suspend all insulin delivery any time you need to be disconnected from your pump (like if you're swimming or bathing) even if it's for a short period of time.



To suspend all insulin delivery:

1. From the Home screen, press the **Up** arrow.
2. Select **Suspend All Delivery**. A confirmation message will appear.
3. Press the **Right** arrow and select **Yes**. The **Delivery Suspended Successfully** confirmation will appear.
4. After suspending insulin delivery, the Home screen will show your insulin delivery is suspended. The pump functions will be limited until basal insulin is resumed.

To resume basal insulin delivery:

1. While insulin is suspended, from the Home screen, press the **Up** arrow.
2. Select **Resume Basal**. A confirmation message will appear.
3. To resume basal insulin, select **Yes**. A **Delivery Resumed Successfully** confirmation message will appear.

Reminder: When using SmartGuard™ technology, the pump adjusts your insulin automatically based on sensor glucose values. Sometimes, the pump stops insulin delivery, even if it's not obvious. You can check your sensor glucose screen to see if this has happened.

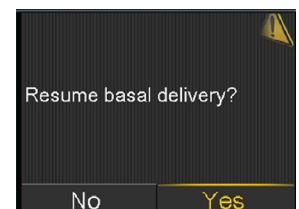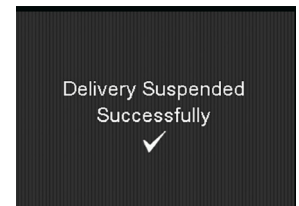

Always seek the advice of your healthcare provider with any questions you may have on your medical condition or treatment.

Important Safety Information for MiniMed™ 780G System:

MiniMed™ 780G System is for type 1 ages 7 and over and type 2 ages 18 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For additional safety information and user guides, see <https://bit.ly/MiniMedRisks>.

US-DBA-2600795 ©2026 Medtronic MiniMed, Inc. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc. The sensor shape and appearance, Abbott, and "a" logo are marks and/or designs of the Abbott group of companies in various territories and used under license. Sensor image © 2026 Abbott. ™*Third-party brands are trademarks of their respective owners.