

# SmartGuard™ Temp Target



The Temp Target feature allows you to temporarily change your SmartGuard target for exercise or other times when less insulin is needed. Always consult a healthcare professional before using Temp Target.

As a reminder, SmartGuard targets are:

100 mg/dL (default)

110 mg/dL

120 mg/dL

## While Temp Target is active:

- The system temporarily targets 150 mg/dL instead of your set SmartGuard target
- No automatic correction boluses are delivered

The duration for the Temp Target can be set from 30 minutes up to 24 hours each time.

## Examples of when to use a Temp Target:



Exercise



Higher activity


(Yard work, cleaning, painting, etc.)



Help prevent lows

## Best practice:

Set the Temp Target 1-2 hours before activity, keep it on during the activity, and stop at the end of the activity.

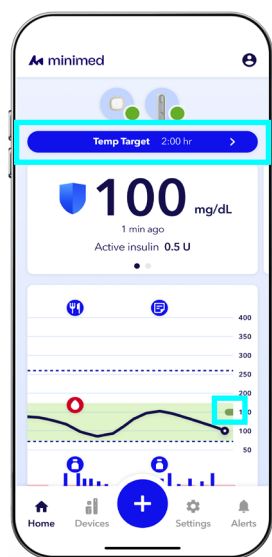

## To set a Temp Target:

1. From the Home screen, tap
2. Select **Temp Target**
3. Tap **Duration**
4. Set the duration
5. Tap **Start temp target**

You will see a blue Temp target banner at the top of the Home screen showing the remaining time and a green bar will start to appear in your Trend graph. When the Temp target time runs out, the banner disappears from the Home screen and the SmartGuard target will be used. The Temp target can also be canceled at any time.

## To cancel a Temp Target:

1. From the Home screen, tap the **Temp target banner**
2. Tap **Stop temp target**

The Temp Target will be canceled, the SmartGuard target will be used and the Home screen will appear with no Temp Target banner.

# Did you know you can suspend your pump?

Use **Suspend All Delivery** if you need to stop all active basal and bolus insulin deliveries. You should suspend all insulin delivery any time you need to be disconnected from your pump (like if you're swimming or bathing) even if it's for a short period of time.

## To suspend all insulin delivery:



1. From the Home screen, tap **+**
2. Select **Suspend all delivery**
3. Tap **Suspend all delivery** to confirm


After suspending insulin delivery, a red Delivery suspended banner will appear on the Home screen. The pump functions will be limited until the basal insulin is resumed.

## To resume all insulin delivery:

1. From the Home screen, tap the red **Delivery suspended banner**
2. Select **Resume basal delivery**

A Basal delivery resumed message will appear briefly at the bottom of the Home screen.

**Reminder:** When using SmartGuard™ mode, the pump adjusts your insulin automatically based on sensor glucose values. Sometimes, the pump stops insulin delivery, even if it's not obvious. You can check your sensor glucose graph to see if this has happened.

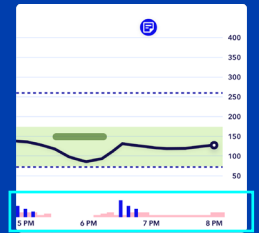

Always seek the advice of your healthcare professional with any questions you may have on your medical condition or treatment.

### Important Safety Information for MiniMed Flex™ System:

“MiniMed Flex™ System is for type 1 ages 7 and over and type 2 ages 18 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For additional safety information and user guides, see <https://bit.ly/MiniMedRisks>.

US-DBA-2600671 ©2026 Medtronic MiniMed, Inc. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc. The sensor shape and appearance, Abbott, and “a” logo are marks and/or designs of the Abbott group of companies in various territories and used under license. Sensor image ©2026 Abbott.™\*Third-party brands are trademarks of their respective owners.