


MiniMed Flex™ System Post-training Guide

Use this as a quick guide for everyday tasks with your MiniMed Flex™ system.

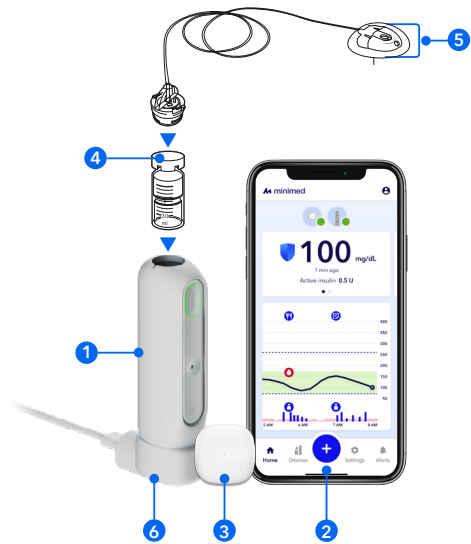

Note: This is an example of what an infusion set looks like as part of the pump delivery system. Your infusion set may look different.

- 1 MiniMed Flex™ insulin pump**
 Delivers insulin into your body through the infusion set.
- 2 MiniMed™ app**
 All-in-one screen and controller for the MiniMed Flex™ pump.
 - You can share your pump information with up to 5 care partners so they can stay informed about your glucose levels and alerts. Care partners can download the CareLink™ Connect app.
- 3 Compatible sensor**
 Measures glucose in the fluid under your skin and sends sensor glucose values to the pump.
- 4 Infusion set**
 Delivers insulin from your pump into your body through a cannula.
- 5 Reservoir**
 Filled with insulin and placed in the pump. Connects to the tubing of the infusion set so that insulin can be delivered to your body.
- 6 MiniMed Flex™ charger and charging cable**
 Charges the pump in about 30 minutes, with a full charge lasting about a week.

How to deliver a bolus

You will use the MiniMed app to deliver bolus insulin for meals and snacks.

1. From the app Home screen, tap .
2. Tap **Bolus calculator**.
3. Enter your glucose, if needed. In SmartGuard™ mode, your sensor glucose reading will be entered automatically, simply tap to use it.
4. Tap **Next**.
5. Enter your carb amount.
6. Tap **Next**.
7. Review and tap **Deliver bolus**.

You will see the bolus delivery status on the Home screen.



How to change your sensor


1. Tap .
2. Tap **Details**.
3. Tap **Change sensor** and follow the instructions in the app.




How to change your reservoir & infusion set

1. From the Home screen, tap .
2. Tap **Change reservoir & infusion set**.
3. Select your infusion set and tap **Start Change**. If your infusion set is not listed, tap **Other** to get started.
4. Follow the instructions in the app to change the reservoir and infusion set.


SmartGuard™ mode

You will know you are in SmartGuard mode by the  displayed on your MiniMed app Home screen. Tap any icon on the sensor graph to view more information. Scroll to view more time.

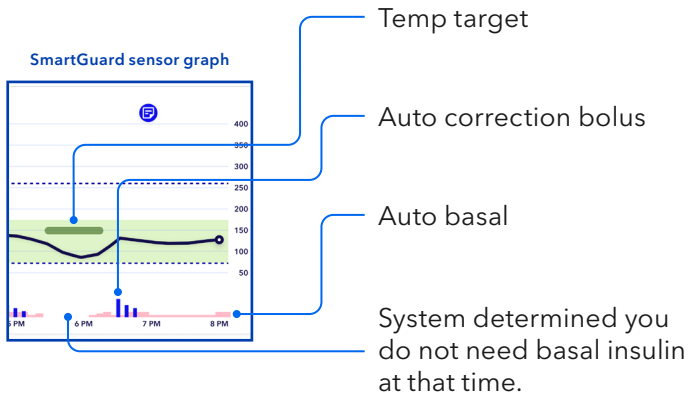


Ongoing care schedule

Here's a quick reference for when to change your set, reservoir, and sensor.


- Infusion Set: Change every ____ days
Fill cannula ____ units
- Reservoir: Change every ____ days
Fill with ____ units
- Sensor: Tap  to see how much time is left on your sensor.

Setting a Temp Target


1. From the MiniMed app Home screen, tap  and tap **Temp Target**.
2. Enter the duration for your Temp Target.
3. Tap Start Temp Target.

A banner appears near the top of the Home screen showing that the Temp Target is on.

Responding to an alert | You must go to the MiniMed app to view and address all alerts.

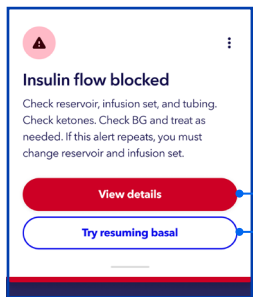

You can tap **View Details** to see more information about the alert here.

 Tap here to take action and respond to the alert.

Note: Some alerts, like glucose alerts, can be snoozed. Tap **Snooze** to silence the alert for a set time. It will sound again if the issue continues.

This is an example of a possible alert in the MiniMed app. Your screen may look different depending on the type of alert happening.

Scan to check out our support websites

MiniMed Flex™ system support

www.minimed.com/en-us/support/product-support/minimed-flex



Sensor support

<https://www.minimed.com/en-us/support/product-support/sensors/>




App manager support

<https://www.minimed.com/en-us/support/app-manager/>



Need help? Call us

 Call our 24-hour Technical Support team at 1-800-646-4633, option 1

Supplies, ordering, replacements, and CareLink™

Orders directly from MiniMed

Call 1-800-646-4633, option 2 (M-F 8a.m.-6p.m. CT)
Order online at Diabetes.shop

Ask about setting up automatic shipments

Request sensor replacements on Diabetes.shop

Username: _____

Password: _____

Orders from a distributor or pharmacy

Name: _____

Phone: _____

CareLink™ account sign-in

Username: _____

Password: _____

Important Safety Information for MiniMed Flex™ System:

"MiniMed Flex™ System is for type 1 ages 7 and over and type 2 ages 18 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For additional safety information and user guides, see <https://bit.ly/MiniMedRisks>.