



# Linking to your patient's CareLink™ Personal account



## Sign into CareLink™ Clinic

Access your clinic data through [carelink.medtronic.com](https://carelink.medtronic.com)

## Is the system user linked to your clinic?

Find their record. You can quickly see whether they are linked.


Patients							CARELINK™ PERSONAL
FLAG	LAST NAME	FIRST NAME	DATE OF BIRTH	PATIENT ID OR MRN	DEVICE	LAST UPLOAD	
<input type="checkbox"/>	test	test	01-01-1969		-		<input type="radio"/> Not linked
<input type="checkbox"/>	test	test	01-01-2000		780G	02-12-2025	<input checked="" type="radio"/> Linked

**If YES:** You can generate a report.

**If YES but no data/old data available:** Ensure the correct account is linked. If the correct account is linked, and data is old, have the user tap Upload now from CareLink in their app's Profile menu.

**If NO:** Use the following steps to link their CareLink™ Personal Account with your CareLink™ Clinic Account.

# Link a CareLink™ Personal Account to CareLink™ Clinic

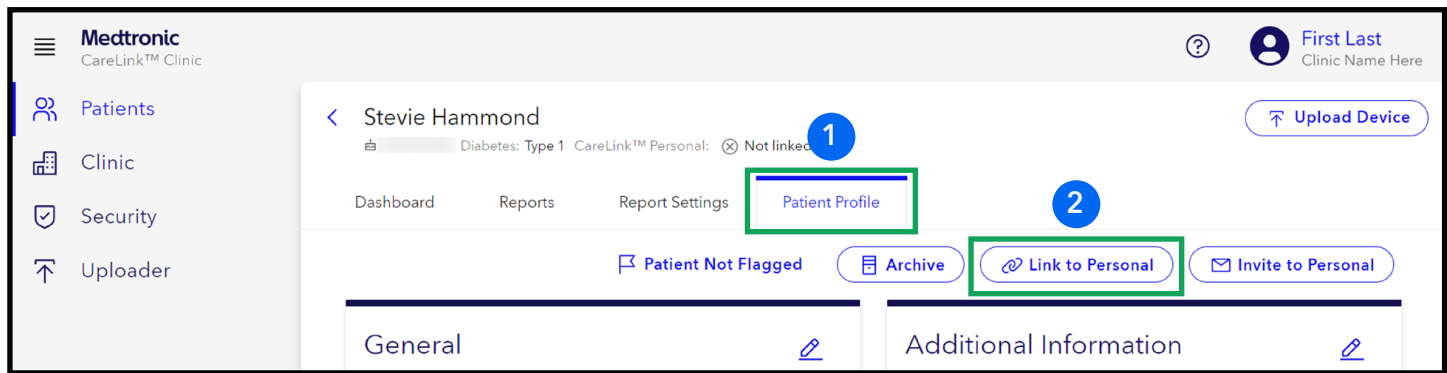

- 1 Log into CareLink™ Clinic and select the record you wish to link. Select **Patient Profile**.
- 2 Click **Link to Personal**.
- 3 Once **Link to Personal** is selected, you can choose how you'd like to link accounts. Patient can enter username and password OR you can select **Generate a unique code to share with patient**.
- 4 If using a unique code, share the unique code with your patient directly or select one of the options for sharing: **Email, SMS Text, or Print**.
- 5 If using the unique code, the patient can enter the code on the **Data Sharing page** available in their CareLink Personal account.  
  
If the unique code was shared via Email, SMS Text, or Print they will receive a message directing them to their CareLink Personal account Data Sharing page to accept the request.

For additional CareLink™ support, please contact our healthcare professional technical support team at 1-800-646-4633, option 9.

**Important safety information for CareLink™ software:** The CareLink™ software is intended for use as a tool to help manage diabetes. The purpose of the software is to take information transmitted from MiniMed insulin delivery systems, glucose meters and continuous glucose monitoring systems, and turn it into CareLink™ reports. The reports provide information that can be used to identify trends and track daily activities—such as carbohydrates consumed, mealtimes, insulin delivery, and glucose readings. NOTE: CareLink™ report data is intended for use as an adjunct in the management of diabetes only and NOT intended to be relied upon by itself. Patients should consult their healthcare providers familiar with the management of diabetes prior to making changes in treatment. For additional safety information and user guides, see <https://bit.ly/MiniMedRisks>.