

# MiniMed™ Simulator App



## A practical tool for healthcare professionals

The MiniMed™ simulator app provides a practice environment that mirrors the look and feel of the MiniMed Flex™ system user interface.


### Use the Simulator app to:

- Gain insight into the user experience with the MiniMed™ app
- Increase familiarity with the MiniMed Flex™ system
- View guided app tutorials
- Provide an opportunity for system users to practice key actions without impacting their therapy

### Get Started

1

**Download** the MiniMed™ Simulator App

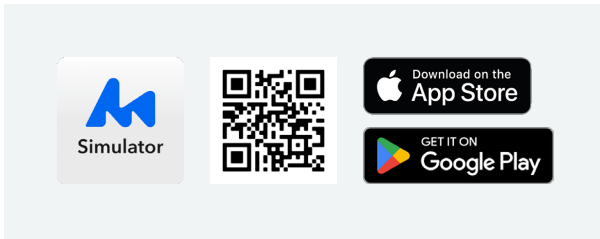


2

**Select** the pump model and sensor using in-app prompts

3

**Explore** the Simulator app

- Review the MiniMed™ app interface
- Explore common user actions in Manual mode and SmartGuard™ mode
- Learn about alerts, tips, and connectivity

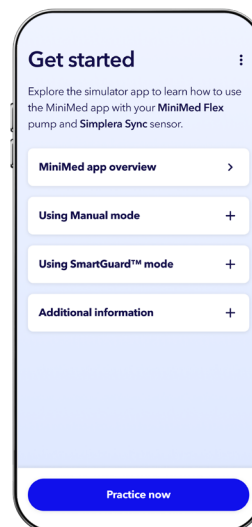

## Practice now

Review how common actions appear and function in the MiniMed™ app.

To start practicing, tap **Practice now** and select Manual mode or SmartGuard™ mode.

## Settings

The Simulator app displays where to find key settings in the app. These include insulin settings, alert and glucose settings, and SmartGuard™ settings.



To find them:


- Tap **Settings**.
- Tap the setting you would like to view.

When using the live MiniMed™ app, this is where specific settings may be selected and adjusted as clinically appropriate.

## Activities for practice

It's time for free play! Below are some key actions MiniMed Flex™ system users may perform everyday in the MiniMed app.

Follow these steps in the Simulator app to practice them:

- Tap .
- Tap the action you would like to practice.

Deliver a bolus

Set a Temp target†

Cancel a Temp target†

Suspend insulin

Resume insulin

Note: The Simulator App reflects the look and navigation of the MiniMed app. Therapy settings are not personalized and do not affect live insulin delivery.

† When in SmartGuard™ mode

### Important Safety Information for MiniMed™ Flex System:

MiniMed Flex™ System is for type 1 ages 7 and over and type 2 ages 18 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For additional safety information and user guides, see <https://bit.ly/MiniMedRisks>.

US-DBA-2600497 ©2026 Medtronic MiniMed, Inc. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc. The sensor shape and appearance, Abbott, and "a" logo are marks and/or designs of the Abbott group of companies in various territories and used under license. Sensor image ©2026 Abbott.™\*Third-party brands are trademarks of their respective owners.