

MiniMed™ 780 system virtual pump



To access the MiniMed™ 780G virtual pump, scan here.

Or visit: medtronicdiabetes.com/minimed-virtual-pump

Program Startup Wizard:

1. Choose preferred language and press
2. Set the time and date using the and and button†
3. Select **Next**, and then **Ok** twice when prompted
4. Click the arrow either below or to the right in the margins and select



† Please note when programming settings and doing exercises, if an item is flashing, it can be edited. If it is not flashing, press Select to allow editing.

Modes available:



SmartGuard technology



Sensor augmented



Standalone pump



Reset virtual pump

Navigation shortcuts

These provide easy access to commonly used functions and can be used from the home screen with or without CGM and in SmartGuard™ technology



Bolus Delivery



Status Screen



Time in Target Range



CareLink™ uploads‡


Press and hold for this function

Note: The virtual pump will time out after ~30 min. Date/time will need to be reset.

‡ This shortcut is for individuals not using the MiniMed™ Mobile app.


Optional button-pushing exercises

Programming the SmartGuard™ target

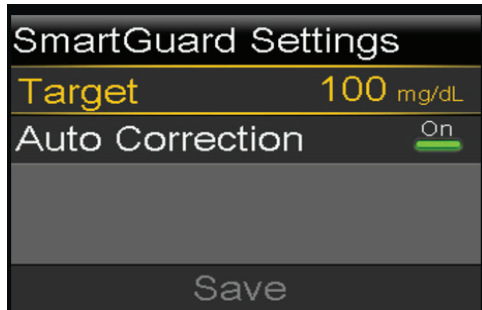


1. Press to open the Menu
2. Press for the menu and press
3. Press to **SmartGuard Settings** and press
4. Press to select **Target**
5. When the target is blinking, use the to adjust to 100 mg/dL
6. Press
7. Press to select **Save** and press

Changing the Active Insulin Time

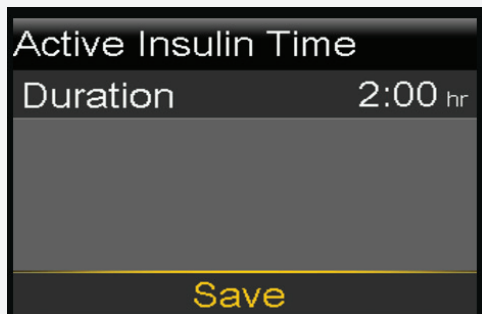

1. Press to open the Menu
2. Press on the
3. Press to **Delivery Settings** and press
4. Select **Bolus Wizard Setup** and press
5. Press to select **Active Insulin Time** and press
6. Press so the time is blinking
7. Decrease to 2:00 by using the
8. Press to confirm
9. Press to select Save and press

Changing the Carb Ratio

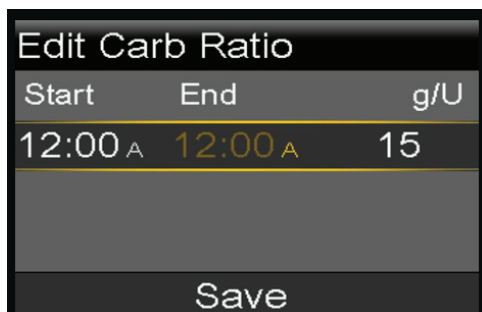

1. Press to open the Menu
2. Press for the menu
3. Press to **Delivery Settings** and press
4. Select **Bolus Wizard Setup** and press
5. Press to select **Carb Ratio** and press
6. Select **Edit** and press
7. Select **12:00A to 12:00A** and press
8. Press when the 12:00A is blinking
9. Decrease the programmed carb ratio by 10-20% (by using the) and press
10. Select Save by pressing

Important safety information – MiniMed™ 780G System is for type 1 ages 7 and over and type 2 ages 18 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For details, please visit <https://www.medtronicdiabetes.com/support/download-library/user-guides> and www.medtronicdiabetes.com/importantsafetyinformationforcompletedetails.

See the device manual for detailed information regarding the instructions for use, indications, contraindications, warnings, precautions, and potential adverse events. For further information, contact your local Medtronic representative.

US-DBA-2501057 ©2025 Medtronic. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc. The sensor shape and appearance, Abbott, and "a" logo are marks and/or designs of the Abbott group of companies in various territories and used under license. Sensor image © 2025 Abbott. ™*Third-party brands are trademarks of their respective owners.

Medtronic