



# Simplifying setting adjustments in SmartGuard™ mode

MiniMed Flex™ system

## Clinician managed settings that impact SmartGuard™ mode

SmartGuard™ target



Active insulin time (AIT)



Insulin-to-carb ratio (ICR)




**Recommended Settings:** 100 mg/dL Target and 2 hour AIT †

## SmartGuard™ Target

The adjustable SmartGuard™ target allows you to choose between the following:

- 100 mg/dL (default and recommended)
- 110 mg/dL
- 120 mg/dL

## Changing the SmartGuard™ target

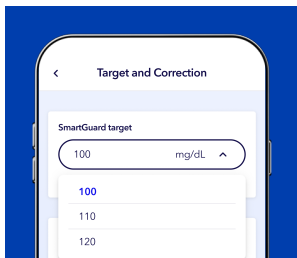


1. From the app Home screen, tap **Settings**.
2. Tap **SmartGuard settings**.
3. Tap **Settings**.
4. Tap **SmartGuard target**.
5. Select the target.
6. Tap **Save**.

## Active insulin time (AIT)

Used as a lever to strengthen auto corrections so the system can step in sooner. The shorter the AIT, the stronger the auto corrections.

### Changing active insulin time

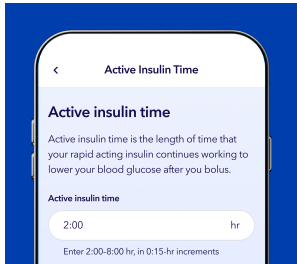

1. From the app Home screen, tap **Settings**.
2. Tap **Insulin settings**.
3. Tap **Active insulin time**.
4. Tap **Active insulin time** box.
5. Scroll to 2:00 hours.
6. Tap **Save**.


## Changing insulin-to-carb ratios (ICR)


Once the SmartGuard™ target and AIT are optimized, only the ICR typically needs ongoing adjustments as needed to support therapy goals.

When bolus habits are appropriate, but Time in Range goals are not being met, consider the following:

- If time above range is >25%: Strengthen (decrease) ICR
- If time below range is >3%: Weaken (increase) ICR

### Changing insulin-to-carb ratios

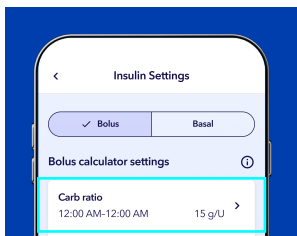

1. From the app Home screen, tap **Settings**.
2. Tap **Insulin settings**.
3. Tap **Carb ratio**.
4. Tap  to change the time segment and carb ratio.
5. Tap **Continue**.
6. Tap **Save**.

† Recommended settings for adults: 100 mg/dL auto basal target and 2 hours AIT. Children 7-14 years old: 110 mg/dL and 2 hours AIT, reduce if no concern with hypoglycemia.

#### Important Safety Information for MiniMed Flex™ System:

MiniMed Flex™ System is for type 1 ages 7 and over and type 2 ages 18 and over. Prescription required. WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day. For additional safety information and user guides, see <https://bit.ly/MiniMedRisks>.

US-DBA-2600520 ©2026 Medtronic MiniMed, Inc. MiniMed and MiniMed logo are trademarks of Medtronic MiniMed, Inc.

™\* Third-party brands are trademarks of their respective owners.